

MyoTrace 400 Reference Guide

Current Mode: Displays the current measurement mode

Battery Level

Connection Status

- No Connection
- Cable Connected
- BlueTooth Disabled
- BlueTooth Searching
- BlueTooth Connected

The function keys are mapped to the menus displayed at the bottom of the screen

Threshold/Scale: Press to adjust the threshold or scale function before using arrow keys during measurement

Press F2 and F3 simultaneously to lock or unlock the keypad

Arrows: Press to adjust Settings during Setup or Threshold and Scale during measurement

View: Press during measurement to change the current view from Bar Graph to Line Graph views or hold for 3 seconds to view firmware version

Audio: Press while in the Channel Setup screen to change Audio Settings

Mode: Press to switch between Continuous, Work/Rest and Template Modes or hold for 3 seconds to change Bluetooth Network