

MRXP Demo Installation Instructions

Thank you for your interest in the MyoResearch XP (MRXP) demo software. To have a successful software installation experience, please follow the three steps listed below to start running the software on your computer. You can access MRXP by either downloading the software from our website at <http://www.noraxon.com> or by requesting a free software demo by contacting us at (480) 443-3413, (800) 365-8985 [US/Canada Only] or via e-mail: info@noraxon.com.

1) A. Web page Installation:

Browse Noraxon's Web page at: <http://www.noraxon.com/products/downloads.php3>
Click the link titled 'Software'

Start the download process by clicking on "**MyoResearch XP Master Edition version x.xx.xx** (current version number, ex: 1.06.54)(**60 MB**), **30-Use Trial**". Fill out the required fields on the form and submit. When you receive the email (check your junk folder if it does not appear in your inbox within 10 minutes), click the link to start downloading and save the setup file to a location either on your desktop or another folder for easy access and reference. Default folder is typically located on your computer's "My Desktop" folder.

Once the setup file is downloaded to your hard drive, locate the setup file and open the program by double clicking on **setup.exe**. Follow the installation wizard, enter the installation password that was included in the email from Noraxon and the installation will continue automatically.

1) B. CD Installation

Insert MRXP demo software CD into your computer's CD drive. A window will pop-up with multiple options. Click "Step 1: Install MRXP". Follow the directions in the installation wizard and enter the password printed on the CD cover when prompted. When completed, click on "Step 2: Install Patch". Another installation wizard will start – this one does not need a password. (Step 2 will not appear in the menu if a patch is not needed.)

After the installation is finished, a new directory will appear under your "All programs" in your Start menu labeled "Noraxon". A folder labeled MRXP Master Trial x.xx will contain three (3) applications: "MRXP 1.06 Application Protocol", "MRXP 1.06 Master Edition" and a third application "Uninstall MRXP 1.06" (if you decide to uninstall the software). Two icons will also appear on your desktop, they are easily identifiable because the icons use Noraxon's logo.

2) Installation of the Video Encoder/Decoder, DivX

MRXP contains a set of video assisted demo data files. To view the EMG synchronized demo data files, you will need to install the **Video Codec DivX 5.0** (or higher) which can be downloaded for no cost from: <http://www.divx.com/divx/>

Ensure that the MRXP software is closed. Execute the DivX installation process by double-clicking on the file. After a successful installation of the DivX video encoder, restart MRXP. There is no need to modify the default settings within the DivX program.

3) Start MyoResearch XP Master Software

MyoResearch XP Master consists of two program versions (2 icons on the desktop):

MRXP Application Protocols

The MRXP Application Protocols program features predefined application protocol groups for measuring and analyzing EMG and other analog signals. The protocols are guided by a wizard that automatically leads you to preconfigured analysis reports. When you click on **Show Info**, located at the bottom right-hand side of the entrance screen, you can gain access to comprehensive protocol descriptions and explanations of each application group. Once you start the **Measure** function, you can view a simulation of a complete application protocol by using the pre-programmed demo data. Under **Database** you can view the (video-based) demo records.

MRXP Master

MRXP Master is the main package of MyoResearch XP. Unlike Application Protocols, it does not use the wizard function; the Master program starts with the main database screen. From the **Database** screen, you can manually access other main menus (i.e. Measure/View/Analysis). In the MRXP Application Protocols version, the settings/features that are available depend on the protocol selected. In the Master version, all software features and numerous setup functions are available.

4) Quick start tutorials and more product information

By downloading the software from our website, you can access "Quick Start" tutorials that help you learn each program version. To access these tutorials, go to <http://www.noraxon.com/products/downloads.php3> and click on the 'Software Manuals' link.

Under the MyoResearch XP heading are the following links

MyoResearch XP - Quick Start Tutorial to MRXP Application Protocols (348 KB)

MyoResearch XP - Quick Start Tutorial (1.00 MB) (Master Package)

You will need to have **Adobe Acrobat Reader** to access these documents. The free software can be downloaded from <http://www.adobe.com>.

Comprehensive **Product Information** is available under the 'Products' tab; select 'Software' to view the MRXP product information.

We hope you have a great experience when using our software. The "demo package" allows you "30 uses", if you need more trial uses, please contact us at (480) 443-3413 or at info@noraxon.com and we will extend your evaluation period. Simply provide us with the **Activation Code** that appears in the small dialog window after the last trial has run.

